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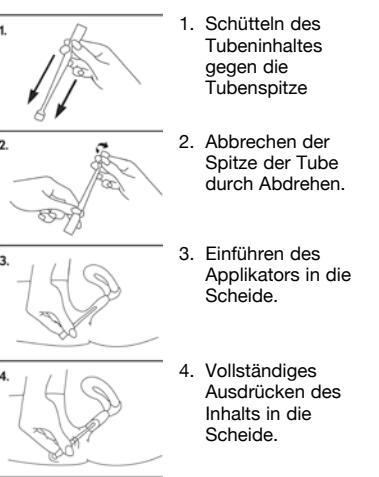
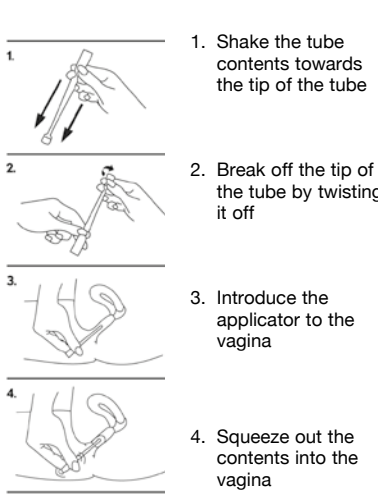
EN
How Gynofit Moisturising Gel works

Gynofit Moisturising Gel is an aqueous vaginal gel that forms a thin film of moisture on the vaginal wall and thus moisturises the mucous membrane.

Intended use
Gynofit Moisturising Gel is used for symptomatic alleviation of vaginal dryness (especially if the mucous membrane is thin, susceptible to attack and vulnerable) and the associated discomfort. Ideal after childbirth, during menopause or cancer therapy.
Furthermore, symptoms that are regarded as irritative, such as burning or itching, are alleviated, thus enhancing physical well-being.
Gynofit Moisturising Gel is also ideal for facilitating sexual intercourse.
Gynofit Moisturising Gel guards against irritation.

Method of use and dose
Each tube contains one application dose. The gel is directly introduced to the vagina from the specially shaped, sealed applicator.

Side effects
When Gynofit Moisturising Gel is used correctly according to the manufacturer's recommendations, its composition is such that no side effects are to be expected beyond an initial but reversible burning sensation and/or itching. In predisposed individuals, especially if the vaginal mucosa is already damaged owing to unsuitable care, hygiene or local contraceptives for example, mucosal irritation or allergic reactions can also occur in isolated cases.

Contraindications
Do not use Gynofit Moisturising Gel if you are known to be allergic to any ingredients; if such reactions are triggered, immediately discontinue using the gel; consult a doctor if necessary.

Use
Depending on needs and the amount of dryness (which is quite different in every woman), use an applicator on a daily basis, ideally during the day.

Notes
Gynofit Moisturising Vaginal Gel can be used for up to 30 consecutive days. There should then be a break of several days before recommencing use.
Gynofit cannot be used for contraception and it does not contain any spermicidal ingredients.
Gynofit Moisturising Gel can be used together with condoms.
Store out of children's reach.
Empty the tubes completely and do not use opened tubes a second time. Do not use damaged applicators.
Store at room temperature (15 - 25 °C). For the use-by date see packaging.

Ingredients:
Aqua, Glycerin, Propylene Glycol, Hydroxyethylcellulose, Citric Acid, Levulinic Acid, Sodium Levulinate, Caprylyl Glycol, P-Anisic Acid, Sodium Hydroxide

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Tentan Deutschland GmbH, Schloss Beuggen 3, 79618 Rheinfelden, Deutschland
Tentan AG, 4452 Itingen